 **Steffca**

**Foldbart Engangsansigtsvisir VI-05**


 **Steffca**

**Disposable Face Shield VI-05**

**STEFFCA®**

## Sådan folder du visiret

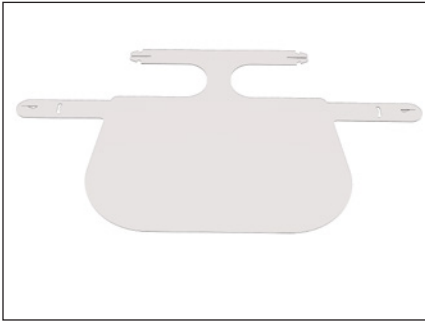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

Mikkel Steffensen  
Eksportchef, indehaver


## How to fold the face shield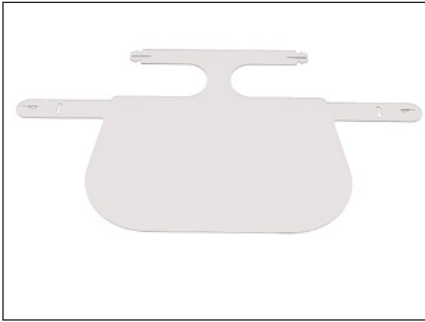

1. Lay the face shield-sheet on a flat surface.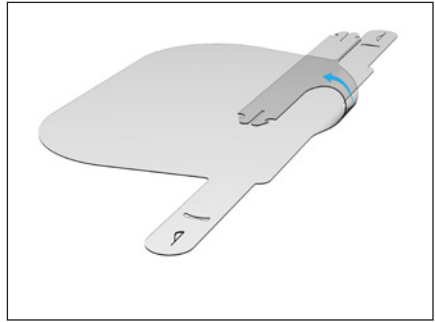

2. Fold over the neck without bending the plastic.

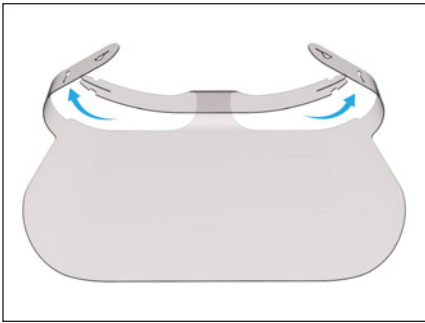3. Align the arms and lock them in the cutouts. Make sure they are securely locked.

4. Insert the fabric elastic band in the holes and tighten the band.

## How to use the face shield

- Grab the face shield and hold the elastic with one hand.
- Grab the top of the face shield with your other hand.
- Pull back the elastic and place the face shield in front of the face.
- Pull the elastic over your head and place it on the back of your head.
- Make sure that the face shield is placed correctly.

Please remember to remove the protective film on both sides

## Correct usage

The face shield protects against drops and splashes of liquids. This reduces the risk of a droplet infection. Due to its design, full protection against viruses, bacteria or other biological agents is only given to a limited extent.

The face shield is disposable and reuse is at the discretion of the user. The face shield does not offer protection against mechanical agents. It can be used with face masks. The face shield can be stored under normal storage conditions (indoors, dry, unheated, or heated storage room). If the face shield is stored correctly, it has a 5-year life span.

The face shield is not sterile upon delivery but can be disinfected prior to usage if necessary. Clean with soap water, disinfect with IPA-alcohol. The face shield cannot be repaired, and no spare parts are available.

## Warnings

Do not use the face shield if it is damaged. Please note that materials that have direct contact with the skin may cause skin irritation for some sensitive users during prolonged use. If this happens, you can initially try to switch to a new shield.

If that does not help, do not use Steffca's face shield until the irritation is gone.

## Marking

The face shield is CE marked: STEFFCA CE  
STEFFCA = Manufacture code  
CE = European Conformity

## Safety

The face shield is CE-marked and meets the Regulation (EU) 2016/425 of the European Parliament and of the Council of 9 March 2016 on personal protective equipment and repealing Council Directive 89/686/EEC.

The shield meets the general requirements of the harmonized standard EN 166:2001 on protective eyewear.


**EU DECLARATION OF CONFORMITY**

Identification of the manufacturer:

STEFFCA A/S  
Romsøvej 24, DK-5800 Nyborg, Denmark

This declaration of conformity is issued under the sole responsibility of the manufacturer to declare that the product:

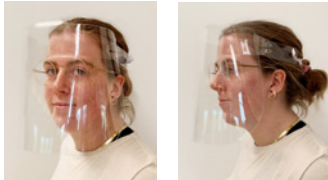**Steffca Single Use Visor, model VI-05**

in risk category:

Risk category II

is in conformity with the relevant Union harmonisation legislation:

Reference no.	Title
2016/425/EU	Regulation (EU) 2016/425 of the European Parliament and of the Council of 9 March 2016 on personal protective equipment and repealing Council Directive 89/686/EEC
2001/95/EC	Directive 2001/95/EC of the European Parliament and of the Council of 3 December 2001 on General Product Safety

and the following harmonised standards:

No.	Title
EN 166:2001	Personal eye-protection – Specifications

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performed the EU type-examination (Module B) and issued EU type-examination certificate XXXX

Signed for and on behalf of STEFFCA A/S

Nyborg, 4 June 2020

Mikkel Steffensen  
Export manager, Owner

# STEFFCA®

STEFFCA A/S

Romsøvej 24 • 5800 Nyborg • Tlf. 6531 3102

E-mail: [info@steffca.dk](mailto:info@steffca.dk) • [www.steffca.dk](http://www.steffca.dk)

STEFFCA A/S

Romsøvej 24 • DK-5800 Nyborg • Tel. +45 6531 3102

[info@steffca.dk](mailto:info@steffca.dk) • [www.steffca.dk](http://www.steffca.dk)